

Era

Deep transdermal application

With Era we tried to optimize the spreading methods to remedy to the limits of the techniques used in the past that is limited concentration of active principle, limited deepness reached in tissues, damaging of tissues due to intensive currents inducted. For these reasons the method represent the state of the art of skin care and treatments, especially in the treatment of cellulite: new and recent studies have been made in different countries with excellent results, representing the future of this new technique that eliminates the use of needles.



Beauty in dentistry



Era Art.1480E




Frame and click!

Use your mobile phone to get further information about this issue.



The results

The most striking results of hyperbaric oxygen therapy are concerned with the significant reduction in wrinkles due to greater skin tone, elasticity and thickness (collagen stimulation).

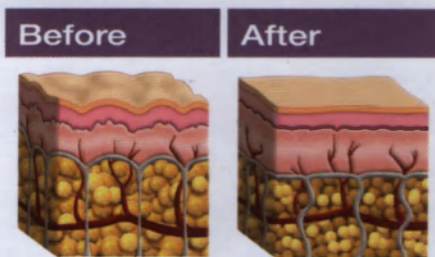

Ask the expert

Who needs this?

All those who want to set out on the path of beauty within the dental industry.

Why Era?

Because it is a device that bio-stimulates the collagen and fills the tissues, by using innovative active ingredients.





The patient's experience is
absolutely pleasurable and without painless

How does it works



Step 1 - Peeling/exfoliation using Bright Skin

Bright Skin, used as a pre-treatment before ERA, prepares the skin rendering it clean and aseptic and ready to receive the active substances in specific treatments. The Bright Skin software includes numerous treatment programmes based on the type of skin (3 types) matching the ultrasound as required to the galvanic currents.



Step 2 - Transportation of the active ingredients using the special handpiece

The ionization chamber is located inside the handpiece and is a real innovation: here, the processes of osmosis and conductivity of the substances gradually take place. Firstly, the active moisturizing substances (in powder form) are mixed in the handpiece with a conductive gel prepared specifically for ERA.



Step 3 - Home treatments to extend the results of the treatment

ERA, a range of products created specifically for electroporation. They contain a combination of active ingredients, well-researched and up-to-date, that prolong and strengthen the results of the treatment. Tecno-Gaz has chosen the best experts in the cosmetics field to offer its clients a rich and effective range of products.

Era

Width	520 mm
Height	1100 mm
Depth	400 mm
Weight	25 Kg
Voltage supply	90 V - 220 V ca.
Working frequency	50/60 Hz
Internal voltage supply	24 Vcc
Max. absorbed power	24 Vcc
Safety class	I BF
Max. voltage output	45 V
Average value current output (R=1K)	20 mA
Working frequency	1760 Hz

Bright Skin

Voltage supply	24 Vcc
Working voltage	5 Vcc
Working frequency	25 KHz
Max. power	0,1 mW

ERA handpiece

Bright Skin (step 1)

Bright skin is an ultrasound device that rejuvenates the skin and prevents ageing of the tissues through the use of ultrasonic waves.

Handpiece for the face (step 2)

- A gradual and non-chaotic transportation of activated substances
- No dispersion and nor spikes in concentration
- Moreover, the handpiece guarantees the high level of hygiene required by health protocols as its stainless steel components can be removed and are all washable and sterilizable.



ERA homecare products

The ERA Kit contains

- Emulsion
- Serum



Beauty in dentistry